

FUSION' 2007 Troubleshooting Guide

Trouble	Causes	Error Indication	Corrective Action
Battery Assembly			
Battery is not being charged	Electronic defect in the Charging Base	Charge Indicator no light; Charge Indicator green light turns on while bypassing the base and directly charging the handpiece from its distal end.	Please contact manufacturer or your dealer to replace Charging Base.
Battery is not being charged	Electronic defect in the Power Adapter	Charge Indicator no light while charging either directly or with Charging Base. Curing light cannot turn on by pressing the On/Off button while plugging adapter cord directly to the handpiece	Please contact manufacturer or your dealer to replace Power Adapter.
Battery is not being charged	Electronic defect in the Battery Assembly	Charge Indicator no light while charging either directly or with Charging Base. Curing light can turn on by pressing the On/Off button while plugging adapter cord directly to the handpiece	Please contact manufacturer or your dealer to replace Battery Assembly.
Handpiece			
Light cannot be activated	The battery is low	Alarm Indicator turns red	Charge the battery. The apparatus can be operated while directly plugging the Power Adapter cord to the handpiece.
Light cannot be activated	The handpiece is overheated	The handpiece head is hot	Allow the handpiece to cool down in ambient or with external air blow to accelerate the cooling.
Light cannot be activated	Bad battery contacts and connection between the Battery Assembly and Main Assembly	Light some times work while unscrew the Battery Assembly from the Main Assembly and then screw them back together	Removing the Battery Assembly from the handpiece by unscrewing it, clean contact pins and battery circular board with a Q-tip slightly dipped in alcohol and then wipe clean with a dry Q-tip, screw the Battery Assembly back tightly to the handpiece.
Light cannot be activated	Electronic defect	Alarm Indicator turns red excluding the above sceneries	Please contact manufacturer or your dealer for service.